EasyCHEE®

POWSTAY PM01A FFP2 NR

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CCQS Certification Services Limited
Block 1 Blanchardstown Corporate Park, Ballycoolin Road,
Blanchardstown, Dublin 15 D15 AKK1
Dublin, Irland

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Particle filtering half mask POWSTAY PM01A of respiratory protection class FFP2

Intended use

This product complies with the requirements of the EN 149:2001 + A1:2009 standard for protection against particles and meets the requirements of the FFP2 protection class therein. It thus offers effective protection against non-volatile solid and liquid particles (dust, smoke, aerosols, aerosol mist).

Respiratory protection is only effective if the protective mask is correctly selected, adjusted and worn during the entire stay in the danger zone.

Checks before use

- Check with a competent person whether this product provides sufficient protection for your application.
- Check the individual packaging of the masks and the masks themselves for possible damage and soiling.
- Check that the storage time has not been exceeded.

Put on and use (Figure 1)

1. Aim the mask with the opening towards you and the nose bridge upwards. Hold the mask by the strap with both hands.
2. Cover your chin completely with the mask. Ensure that the nose clip is pressed against your nasal bone.
3. Pull the loops over your ears. You can also attach them to the back of your head with the adjustable clip.
4. Adjust the nose clip to the shape of your nose and face by pressing and bending with both hands.

How to use the mask

- The respirator should only be removed from its individual packaging immediately before use.
- It cannot be used on deep scars or if there is a lot of hair around the edge of the mask.
- The mask should be worn correctly during all presence in the danger zone and replaced if necessary.
- If the instructions for use are not followed, the health of the user may be affected and long-term damage may occur.
- This product marked "NR" is not reusable, has a maximum usage time of approximately eight hours and must not be used again after the end of a work shift.
- The respirator must never be modified or repaired.

Limits of use

The mask is not suitable for protection against harmful radiation, gases and vapours. It cannot be used in anoxic environments, underwater, fire fighting, etc. The product cannot be cleaned and reused. If the air is highly polluted, if you experience dizziness or tightness in the chest, or difficulty breathing, please leave the area of use immediately and remove the mask.

Storage and disposal

The following storage instructions apply to the packed product:

- Ambient temperature: -20°C to +40°C
 - Relative humidity: up to a maximum of 80%
 - Keep away from dirt, dust and direct sunlight.
- Used respirators should be disposed of as hazardous waste, if possible in accordance with local regulations.

Meaning of the pictograms on the packaging (Figure 2)

- A: See the manufacturer's information leaflet (this document)
B: Temperature range of storage conditions
C: Maximum relative humidity of storage conditions
D: end of storage period (expiry date); date format: year/month

Approvals, certificates

This product meets the requirements of the European PPE regulation 2016/425 and therefore carries a CE mark.

The type examination certificate was issued by:

CCQS Certification Services Limited
Block 1 Blanchardstown Corporate Park, Ballycoolin Road,
Blanchardstown, Dublin 15 D15 AKK1
Dublin, Ireland

Notified body identification number: 2834


The complete EU Declaration of Conformity and further information on the product can be found on the website www.easychee.com.

THIS IS PERSONAL PROTECTIVE EQUIPMENT,
NOT A MEDICAL DEVICE!


Bild 1:
Fig. 1:

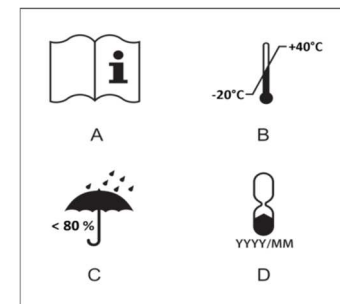Bild 2:
Fig. 2:

Distribution, Info & Support:

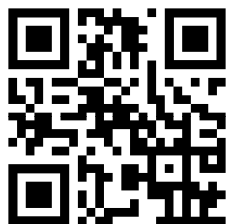EasyCHEE.com

Hersteller · Manufacturer · Fabricant
Produttore · Fabricante · Fabrikant:

Shenzhen Powerqi Technology Co.,Ltd.
2nd & 3rd Floor, Building A4, Block A,
Fangxing Science & Tech. Park,
No. 13 of BaoNan Road,
Longgang District, Shenzhen, China
Tel. +86-0755-84522599

Import und Vertrieb · Import and distribution
Importation et distribution · Importazione e distribuzione
Importación y distribución · Import en distributie:

EasyCHEE®
Dipl.-Ing. Vyngintas Krasauskas
Birnauer Str. 10
80809 Munich, Germany
<https://easychee.com/>
Tel. +49 160 4 002 002

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